


Contour LINK


WIRELESS BLOOD GLUCOSE MONITORING SYSTEM

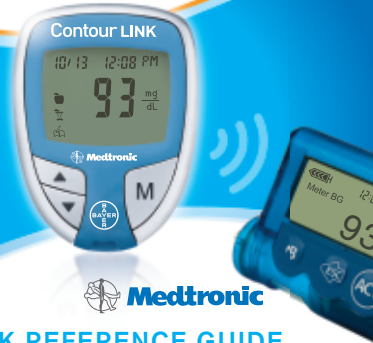


NO CODING

Medtronic


QUICK REFERENCE GUIDE



Your **CONTOUR® LINK** Meter:

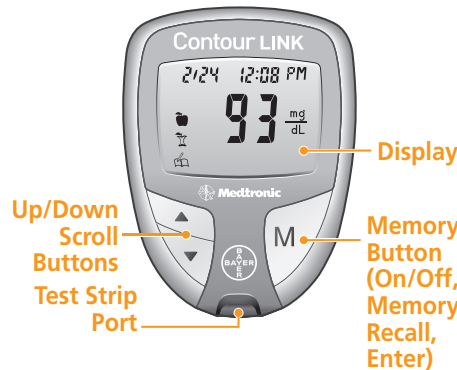


Your **CONTOUR®** Test Strip:

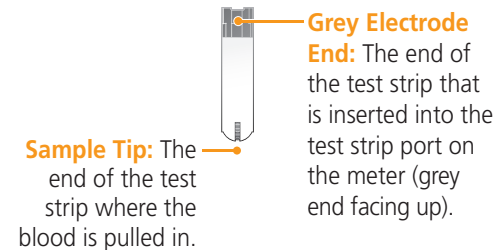


TESTING YOUR BLOOD

*Before starting, be sure the **Send** option on the meter is set to **On** (the **Send** arrow is displayed to the right of the date in the display window). Check battery status and wash your hands before testing.*

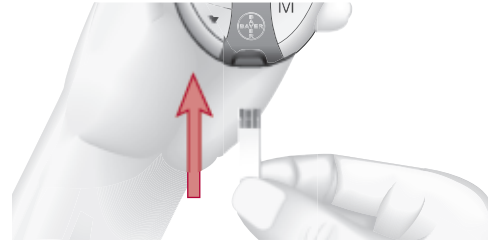


1. **Hold the test strip** with the grey end facing up. Insert the grey end of the test strip into the test strip port.

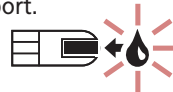

The meter will turn on and a test strip with a flashing blood drop will let you know the meter is ready to test.

2. **Obtain the blood drop.** See your User Guide or Bayer's **MICROLET®2** insert for instructions.

3. **Immediately touch the tip** of the test strip to the drop of blood. The blood is pulled into the test strip through the tip.

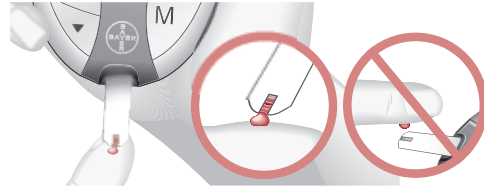

Keep the strip in the blood drop until the meter beeps.



Your result is displayed after a five-second countdown. It will flash to indicate that it is being transmitted to your compatible Medtronic device.

Important: always check the display on your Medtronic device to ensure that the glucose result shown agrees with the meter display.

To set up the link feature on your Medtronic device, please refer to the Medtronic device addendum located in your device box.

To **stop transmission of a result** (if not accepted by the Medtronic device within the first 5 seconds), press **M**. The transmission will be cancelled. (The result cannot be re-sent later.)

For **marker instructions**, see **other side**. You must mark your results while the result is still being displayed (**before** you remove the test strip or turn the meter off). The markers will be displayed after the result has been sent (or the transmission cancelled).







TURNING OFF THE **SEND** OPTION FOR A SINGLE RESULT

If you do **NOT** want to send a blood test result to your Medtronic device, you can turn off the **Send** option for a single test.

1. Insert the test strip to turn the meter on. **Before you begin your test**, press and hold the **▲** or **▼** key until "**OFF**" is displayed. The **↑** displayed next to the date will disappear.
2. Test your blood as usual. The result will not flash, indicating that it is not being sent.
3. When you turn the meter on to perform another test, the **Send** option will again be on.

See your user guide for a complete list of error codes and symbols.

Need Help? Bayer Diabetes Care Customer Service: 1-800-348-8100 (available 24 hours a day, 7 days a week).

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
 (will remain on display)	Low batteries.	Replace the batteries as soon as possible to avoid the risk of a test result not being stored in the meter.
 (will flash for 10 seconds then the meter will turn off)	Dead batteries.	Replace the batteries immediately to avoid the risk of a test result not being stored in the meter.
E1 	Temperature out of range.	Move to an area that is within the operating range for the meter (41°F – 113°F). Let the meter adjust to this temperature for 20 minutes before testing.
E2 	The test strip has not filled enough for an accurate test.	Remove the test strip and retest using a new test strip.
E3 	The meter is sensing a used test strip or blood was applied before the meter was ready.	Remove the test strip and retest using a new test strip. Wait until you see the flashing blood drop in the display before adding the blood sample.
E4 or E7 	Test strip not inserted correctly or using the wrong test strip.	Remove and reinsert the test strip properly. Only use CONTOUR Test Strips.
HI or LO	HI = Test result is above 600 mg/dL; LO = Test result is below 10 mg/dL.	Wash and dry your hands and the test site. Repeat the test using a new test strip. If your result is still " HI " or " LO ," contact your physician or health care professional immediately.

WARNING: Glucose levels above 250 mg/dL or below 50 mg/dL may indicate a potentially serious medical condition.

Test result continues to flash for 60 seconds, then stops.	The Medtronic device did not acknowledge receipt of the test result within 60 seconds.	<ul style="list-style-type: none"> • Be sure the Medtronic device is located within 4 feet of the CONTOUR LINK meter. • Check that the Medtronic device is set to accept meter results and the meter ID has been programmed into the device. • Check the display on the Medtronic device to determine if the glucose result has been received by the device. If it has not, you can enter the test result manually. <p>If you continue to have problems, contact Bayer Diabetes Care Customer Service.</p>
--	--	--

MARKERS AND ALARM



See your user guide for complete instructions.

Pre-meal

Marks a test result that was taken before a meal.

Post-meal

Marks a test result that was taken after a meal.

Logbook

Marks any result as being unique or different in some way.

TO NOT MARK RESULTS

Press **M** or remove used test strip.
OR
 Press **▲** or **▼** until all three markers disappear.
 Press **M** and remove used test strip.

TO MARK RESULTS

YOU WILL SEE

TO SET

YOU WILL SEE

WHAT IT MEANS

Pre-Meal
 Press **▲** or **▼**



Press **M**




Result is marked as a pre-meal test. You can now set a test reminder alarm.

Alarm
 Press **▲** or **▼**



Press **M**

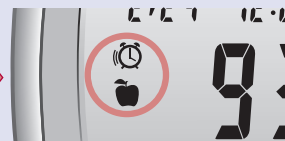

Alarm is set to go off in the time you selected (1.0 to 2.5 hours) to remind you to run a post-meal test.

Post-Meal
 Press **▲** or **▼**



Press **M**



Result is marked as a post-meal test.

Logbook
 Press **▲** or **▼**



Press **M**




Result is marked in memory as unique. You should note this in your Logbook.

WHEN ALARM SOUNDS

YOU WILL SEE

- Meter turns on automatically.
- Press any button once to silence the alarm and twice to turn the meter off.
- Run another blood test and mark it with a (post-meal marker) (see chart at left).

Your pre-meal blood test result.

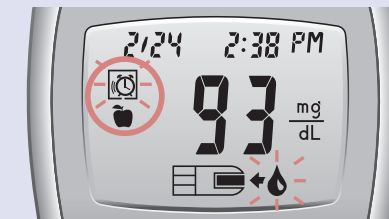

TO REVIEW RESULTS

Press **M** – the 14 day average will appear on the display.
 Press **▲** to view the 7 day total.
 Press **▼** to review previously stored results.
 Press **▲** or **▼** to scroll forward and backward through results. Either press and release the button to view results one at a time or hold it to move through results continuously.
 When **End** appears on the display, you have viewed all results in the memory.
 Press **M** to turn the meter off.
 For complete instructions for use, including cleaning and disinfecting your meter and lancing device, please see the meter user guide.

WARNING

- The meter and lancing device are for single patient use. Do not share them with anyone including other family members! Do not use on multiple patients!
- All parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after you have performed cleaning and disinfection.



Bayer HealthCare

Bayer HealthCare LLC
 Mishawaka, IN 46544 USA

Need Help?

Bayer Diabetes Care

Customer Service: 1-800-348-8100
 (available 24 hours a day, 7 days a week)

www.bayerdiabetes.com/us

Bayer, the Bayer Cross, *CONTOUR*, *MICROLET* and the No Coding logo are registered trademarks of Bayer. All other trademarks are the property of their respective owners.

Bayer

84267113 © 2011 Bayer.
 All rights reserved.

Rev. 12/11